

Pet Home Tests

Companion Animals

●Dogs ●Cats ●Rabbits ●Ferrets ●Hamsters ●Hedgehogs

Fecal Testing For:

●Roundworms ●Whipworms ●Coccidia ●Hookworms

As well as Giardia* & fecal occult blood*

Worm Test = Purchase price + postage

***Giardia ELISA SNAP test** = Purchase price + \$30

***Fecal Occult Blood test** = purchase price + \$20

Our at home tests will identify parasites without the cost and time of a vet appointment.

Purchase this kit, collect the sample, and mail for processing. We will notify you of the results by email within 24 hours of receipt.

Safe time and \$\$\$ with at-home veterinary tests!

- All pets are susceptible to worms, especially puppies and kittens.
- Some worms can be transferred to humans.
- Signs indicating worms are diarrhea and lethargy
- Our at home test products were developed by a veterinarian and have been in use since 1980.
- Save up to 50% on our tests and wormers compared to at the vet.

Directions for Worm Tests

1. Print contact information CLEARLY on the adjacent Laboratory Request form. We must be able to read your information to process the test.
2. Label the provided zip-lock bags with your pet's name.
3. Place 1-2 tsp of fecal matter collected within the last 24 hours into one of the small zip-lock bags and seal.
4. Place that bag into the 2nd zip-lock bag and seal.
5. Place the small bags into the large bag and seal.
6. Place the large bag AND this entire original 3-page form into the envelope.
7. Apply postage and press flat so that the package will pass through postage machines.
 - The post office will not handle and packages that are not securely sealed and dry. There is a \$10 charge to re-send the forms if they are not properly sealed and legible.

- You may send multiple samples inside a larger mailer to save money, but you must provide an original 3-page completed form for each sample.
- If you see worms in your pet's stools, take a picture and enclose it or email it to us at pethometests@gmail.com.