

# Restore Veterinary Gel

**Noxsano®**

**Restore Gel by Noxano** helps veterinarians speed or restart wound healing after injury, surgery or disease, to rapidly return patients to full health. Rapid healing reduces risk of infection, number of dressing changes, and clinic workload. This decreases stress of the animal and owner while lowering treatment cost.

**Indications:** Recommended for infection control and treatment of all partial and full-thickness wounds, including surgical incisions in small and large animals.

1. Open Pouch, remove syringe.
2. Remove cap & fill to 5ml w/ water.
3. Recap syringe & shake vigorously.
4. Liberally apply gel directly to wound.

**Restore Gel by Noxsano** must be activated prior to use. Allow time for gel to set before application. **For Veterinary use only.**