

# Hoof Tite™ Equine Fast Set

## *Chem Select*

(30 sec)

Formulated For Use In Reconstructing Hooves, Filling Chips & Cracks & Adhering Shoes

220 mL

**CLEAR**

FOR PROFESSIONAL USE ONLY!  
KEEP OUT OF REACH OF CHILDREN

**Before Use:** Refer to MSDS for all handling and safety precautions.

**Directions For Use:** Insert cartridge into dispenser/gun. Untwist round cap and remove dual plug. Place mixer tip into place. Slide cap down around the mixer tip and twist cap to secure. Dispense through gun.

**Preserving Unused Product:** Replace dual plug and secure with cap.

Apply thin layer of Hoof Tite & scuff metal prior to applying shoes.

Product temperature between 55° - 78° F for optimum performance.

Use cooler or warmer box if necessary.

### **FIRST AID**

#### **Inhalation**

If inhaled, move victim to fresh air. Keep victim at rest and keep warm. In case a person may have been poisoned, contact a physician immediately.

#### **Eyes:**

Immediately flush eyes with plenty of flowing water for 10-15 minutes holding eyelids apart. Consult a physician.

#### **Skin:**

Wash immediately with water and soap. Consult a physician if irritation persists.

#### **Ingestion:**

Caution if victim vomits; there is RISK OF ASPIRATION! Immediate medical attention is required.

### **CONTENTS:**

#### **Part A (Resin)**

CAS-No. 9082-00-2 Polyolpolyether

CAS-No. 64852-22-B Polyoxialkylamin

#### **Part B (Hardener)**

CAS-No. 101-68-8 4,4' methylenediphenyl dilsocyanate, dipenylmethane-4,4' dilsocyanate

CAS-No. 64-02-8 Ethylendiamineteraessigsäure-Tetranatriumsatz

CAS-No. 9016-87-9 (veraltet) Diphenylmethandiliso-cyanate, siomeren und Homologen  
CAS-No. 5873-54-1 o-(p-isocyanatobenzyl) phenyl isocyante, diphenylmethane-2,4'-  
dilsocyanate

Proudly Made in the USA